

Instructions for use**Paraffin Wax****【Directions for Use】**

1. A plate of wax is heated with a flame and if necessary, the sheet is pressed onto the model to create the denture base.
2. When moderately softened, roll it up to form a bar.
3. The bar-shaped wax is made into a horseshoe shape and pressed against the base plate.
4. Over and under parts are added or trimmed to complete the bite plate.
5. Adjustments of the occlusal denture base, arrangement of artificial teeth, and fabrication of the wax denture are performed in accordance with the common practice.

【Caution】

1. When incinerating this material, install a local exhaust system, ventilation fan, etc. and avoid working in a closed room. Do not inhale smoke and vapor generated by incineration.
2. If any problems such as foreign matters are mixed in before or during the use of this material, discontinue use.
3. When using this material, be careful not to get burned.

【Important Note】

1. Do not use for patients with hypersensitivity such as rashes or dermatitis to this material.
2. A user who has a rash, dermatitis or other hypersensitivity to this material should wear gloves and avoid direct contact with this material.
3. Patients who develop hypersensitivity symptoms such as rash, eczema, redness, ulcer, swelling, itch and numbness due to the use of this material should discontinue use and consult a doctor.
4. A user who develops hypersensitivity symptoms such as rash, eczema, redness, ulceration, swelling, itching, or numbness due to the use of this material should stop using it and seek medical advice.
5. Advise users and patients that any serious accidents involving the device must be reported to the manufacturer and to the competent authorities of the Member State in which the user or patient resides.

【Storage】

- Store in a cool, dark place.
- Do not leave the product in a warm place (e.g., near a stove or in direct sunlight).
- The use and storage area shall be equipped with a fire extinguishing system.
- Store in a place where fire is strictly prohibited and do not store in large quantities in one storage area.
- Avoid direct sunlight during storage and transportation.
- Properly store and manage so that only authorized personnel can access.

【Expiry date】

There is no definite expiration date, but it should be used within 10 years from the date of manufacture.

【Disposal information】


Remaining stocks must be disposed of according to the corresponding national legal requirements.

【Potential Hazard】

This product may cause allergic reactions such as rashes and dermatitis.

**【Manufactured by:】**

 **YAMAHACHI DENTAL MFG. CO.**
 54-1 Ochigara, Nishiura-Cho, Gamagori, Aichi-Pref, 443-0105, JAPAN
 Tel: (+81) 533-57-7121 / Fax: (+81) 533-57-1764
 E-mail: box@yamahachi-dental.co.jp;
 Web: www.yamahachi-dental.co.jp

 **OBELIS S.A.**
 Bd. Général Wahis, 53, 1030 Brussels, Belgium