

# GLOSSY COAT

Light Curing Glossing agent



Time-consuming and labor-intensive work of final glossing can be finished quickly.

For Denture finishing



For Crown/Bridge finishing



## Step. 1

After silicone polishing, clean and dry the glossing part.




## Step. 2

Apply the Glossy Coat.



## Step. 3

Apply the light curing.

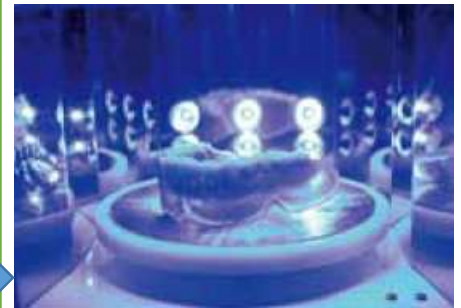


YAMAHACHI DENTAL

# GLOSSY COAT

## Colorant and Abrasion Resistance

High performance in colorant and abrasion resistance after curing.

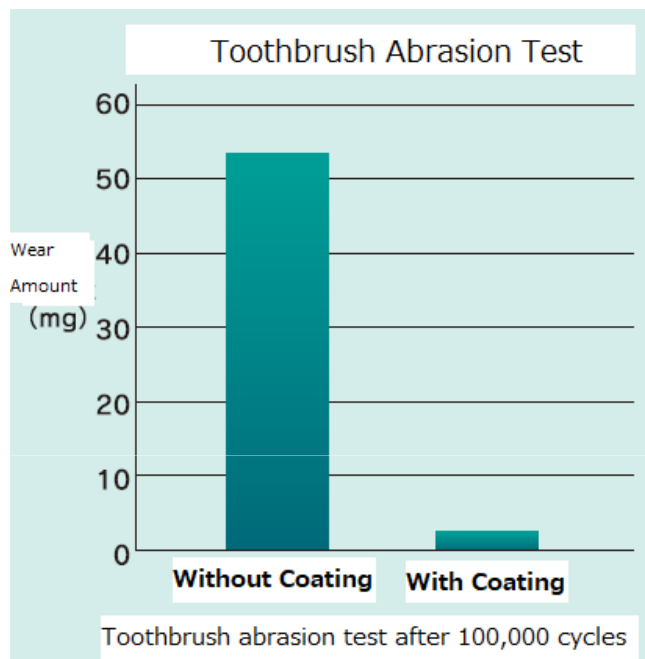

## Characteristics

- Surface glossing agent of Light Curing type with Resistance for Colorant and abrasion.

- Good elongation and thin application allows for a thin film thickness of agent.

- Very little stickiness after curing

- Demonstration Video (Japanese only)

<https://www.youtube.com/watch?v=PNkepA8OSpU>

Description	GLOSSY COAT
Package	15ml / pc
Wavelength	365 - 410 nm



**YAMAHACHI DENTAL MFG., CO.**

54-1, Ochigara, Nishiura-Cho, Gamagori City,  
Aichi Pref., JAPAN - 443-0105

